



BPL Air Mattress



Happier Living Everyday

Therapy for Bed Sores



High Performance Motor

Pure copper compressor has excellent heat radiation and extremely silent during operation, therefore increasing the lifetime of the device.



Extra End Flaps

Extra End Flaps on the ends gives provision to tuck-in the mattress with the bed for restricted movement during use.



Alternating Pressure Therapy

Continuously and sequentially inflate and deflate air bubbles to avoid long term pressurization of tissue, while stimulating blood circulation.



Pressure Indicating Knob

It helps in facilitating optimized pressure setting depending on the user.



Superior Quality Mattress

Medical grade PVC mattress makes it highly elastic and durable.



Auto-Adjusting Air Bubble

Automatically adjusts the internal pressure of the air bubble in response to user body weight, maintaining high level of comfort & optimal pressure redistribution.



Low Noise Operation

Enhanced product design provides quite therapy, making it ideal for home & hospital use.




Anti-Vibration Metal hooks

Metal hooks are designed to make it suitable for most bed types, providing a compact fit and restricting vibrations.




BPL Air Mattress - Alternating Pressure Therapy


BPL Air Mattress is made from inflatable air cells and the pump comes with double air vents. This alternating pressure redistribution system aids in prevention and treatment of bed sore. Air cells would be automatically inflated and deflated alternately through the consistent cyclic airflow to create a therapeutic sleeping surface, which helps optimize weight redistribution and reduce pressure spots.

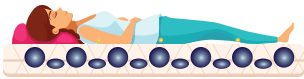

The first group of air cells is inflated, and the second group is deflated

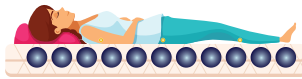

All the air cells are inflated.

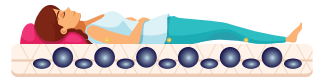

The first group of air cells is deflated, & the second group is inflated.

Product Specifications

Product Description	Medical Air Mattress
Model	BPL Air Mattress
Power Source	AC 220V 50Hz
Cycle Time	12 Mins
Air Flow	≥3 L/Min
Pressure Range	50-110 mmHg
Mattress Type	Medical Grade PVC, 130 bubbles with End Flaps
Mattress PVC Thickness	0.3 mm
Maximum Weight Capacity	113 Kgs
Dimension (Pump)	23 x 11 x 10 cms
Dimension (Mattress)	200 x 90 x 7 cms
Storage Environment	Temperature: 5°C to 40°C (41°F to 104°F)

*Technical specification subject to change

ISO 13485:2016 CERTIFIED COMPANY
ISO 9001:2015 CERTIFIED COMPANY

BPL MEDICAL TECHNOLOGIES PRIVATE LIMITED

Regd. Office: 11th KM, Bannerghatta Road,
 Arakere, Bangalore - 560076, India.

Toll Free: 1800-4252355

Website: www.bplmedicaltechnologies.com

For Enquiries: sales.medical@bpl.in

CIN: U33110KA2012PTC067282

Good at Heart[®]
 A BPL Medical Technologies Initiative

Wake up to Wellness[®]
 A BPL Medical Technologies Initiative

THE ECONOMIC TIMES



BPL
home care[®]
 Made Easy and Reliable